

Instruction For Use (IFU)

Filter Cartridge (FC)

O.N.: 2165-10 REF PN: 01031254

O.N.: 2164-10 REF PN: 01031327

O.N.: 2163-10 REF PN: 01031279

O.N.: 2167-10 REF P/N 01031569

O.N.: 2166-10 REF P/N 01031593



Warning! This is a part of the MAXAIR System User's Instructions, P/N 03521015. Read complete User's Instructions before use.

Bio-Medical Devices Intl • www.maxair-systems.com • 17171 Daimler Street Irvine, CA 92614 • 800.443.3842



CAUTION

- Ensure that the black FC-to-Helmet gasket seal is uniform and flat continuously around the Filter Cartridge.
- Helmet and Filter Media is exposed. Handle with care. Avoid contact with filter media.

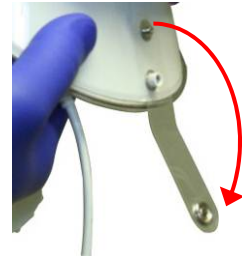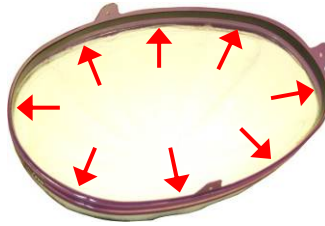

NOTE:

Before starting, FC assembly, ensure that the FCC Pull Tab on the Helmet is unsnapped (Optional, not on all helmets.)

Installation:




1. Hold the Helmet upright with one hand, and with the other hand align the FC with the Helmet rear post.



2. Rear Snap: Snap and secure the FC rear tab in place onto the Helmet rear post.



3. Center and lift the left and right side FC tab such that it sit on top of the side post.

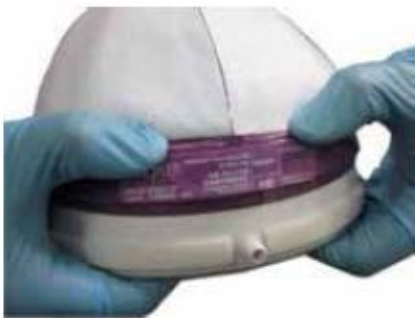

4. Using the step on the FC front, push down the FC firmly until it cannot go down any more.



5. Repeat for each side such that the side tabs are in line with the Helmet side post.




6. For Left and Right Snap: Repeat step number 5 and follow by sliding the FC hole over the post.

Uninstallation



1. Lift and unsnap the FC Left and Right side.

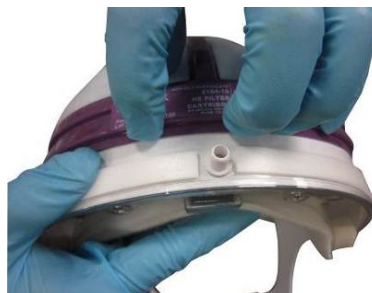

2. Using the bottom step on the FC front, uncap away from the Helmet.



3. Continue the motion to unsnap the FC rear tab away from the Helmet.