


Before use, refer to User's Instructions, P/N 03521015, for detailed System instructions and information.

**NOTE:** Four Helmet Mounting Snaps (arrows at right) attach the Helmet Liner to the Helmet.

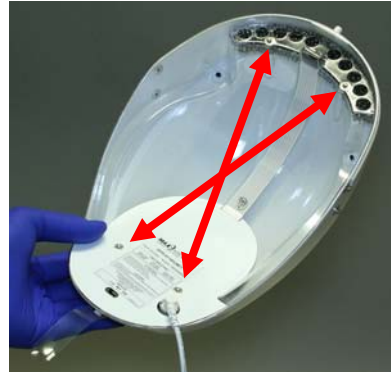

1. Position the Liner near the Helmet inside (A). The Liner Cord Slot (red arrow) allows the Liner to install with the Power Cord attached to the Helmet. Place the Liner between the Power Cord and the Helmet.



A

2. Secure the Liner to the Helmet. Begin by securing the Liner front edge under and against the front lip of the Helmet (B1, red arrow). The Liner front should be "caught" in this recessed area at the Helmet front.



B1

3. Secure one front Liner Snap to the Helmet Snap by pressing the Liner Snap firmly to the Helmet Snap. A click occurs when the snap is secured. Repeat for the other front Liner Snap. Repeat for each rear Liner Snap (B2).

B2

Top Left



Top Right




Bottom Left



Bottom Right



- Secure the Helmet Liner rear (C1) to the Helmet rear (C2) by pressing them together. A slight snap occurs when the Liner is secure to the Helmet (C, press/squeeze in direction of red arrows).

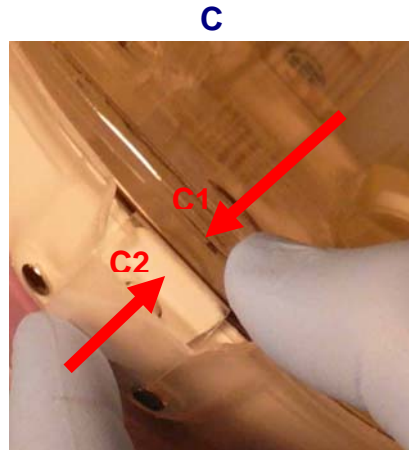

- Ensure the Helmet Power Cord is secure to the Helmet.

If necessary, place the short right-angle Power Cord connector end into the connector on the Helmet Controller Board. Press in firmly to seat the power cord connector securely at the bottom of the Helmet Connector (D).

