



Before use, refer to User's Instructions, P/N 03521015, for detailed System instructions and information.




NOTE: Before starting Filter Cartridge assembly, ensure that the FCC Pull Tab on the helmet is unsnapped (red arrow at right).

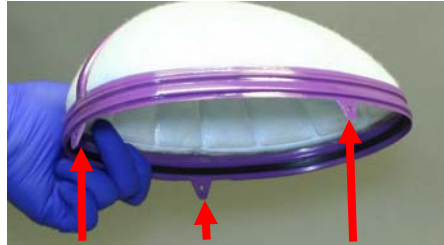

NOTE: Three Mounting Snaps attach the Filter Cartridge to the Helmet (red arrows to the right).

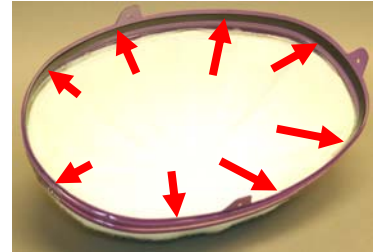




NOTE: Ensure that the black Filter Cartridge-to-Helmet gasket seal is uniform and flat continuously around the Filter Cartridge.

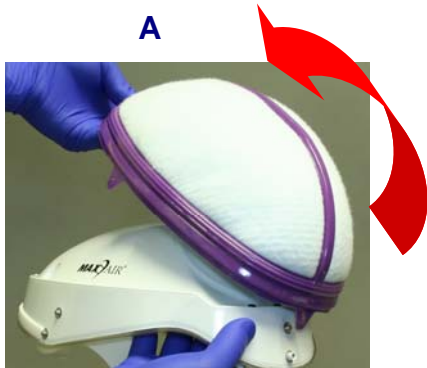

1. With Helmet Liner installed, hold the rear of Helmet upright with one hand, and with the other hand cap the Filter Cartridge over the Helmet from rear to front (A, follow curved arrow).



2. Fully secure the Filter Cartridge on the Helmet. Start at the front and go all around the Filter Cartridge. Use the flat surface around the Filter Cartridge, and carefully push it downward firmly until it meets with resistance and all snap tabs are in line with Helmet snaps (B).

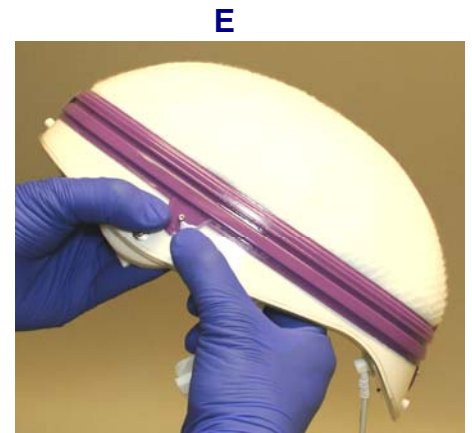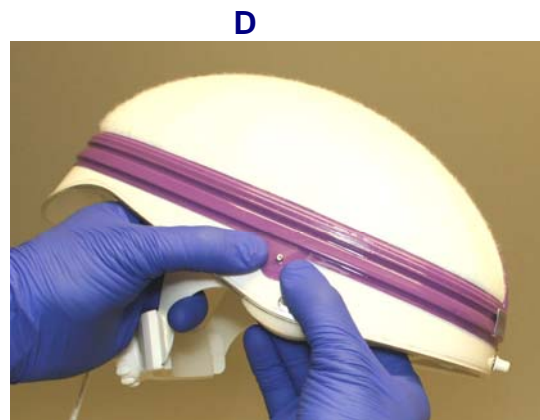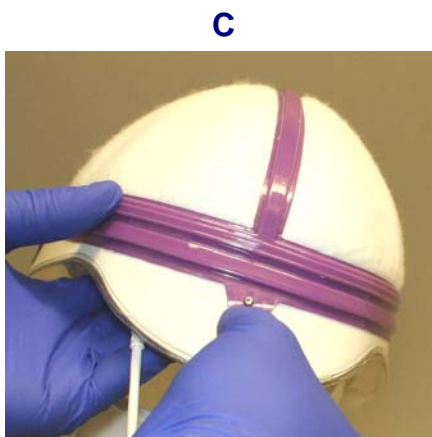

3. Pull the Filter Cartridge Snap Tabs over the Helmet Snaps and secure them together, starting with the rear (C), then one side (D), then the other side (E).